
EASY-TAG™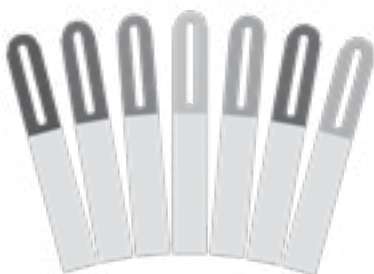

EC REP MedEnvoy Global B.V.
Prinses Margrietplantsoen 33
Suite 123
2595 AM The Hague
The Netherlands

SCANLAN ★

INTERNATIONAL

Surgical products designed and manufactured by the Scanlan family since 1921

SCANLAN INTERNATIONAL, INC.
ONE SCANLAN PLAZA
SAINT PAUL, MINNESOTA 55107 U.S.A.
800-328-9458 • 651-298-0997
FAX 651-298-0018
www.scanlaninternational.com

EASY-TAG™ Tracking System

Clinical Benefits:

The EASY-TAG™ Tracking System is not intended for patient contact or use. The EASY-TAG™ Tracking System is a single-use device to provide users with a means of identifying instruments' processing status or the contents of sterilization trays and allows placement of pre-printed labels or writing on both sides of the tag. EASY-TAG™ Tracking Systems come in three different sizes, in multiple colors, and in blank or preprinted configurations. EASY-TAG™ Tracking Systems are intended to be used with steam, ethylene oxide (EO), or vaporized hydrogen peroxide (gas plasma) sterilization modalities.

Intended user

The intended users of the EASY-TAG™ Tracking System are Sterile Processing Department Personnel and associated areas in the health care facility setting.

Intended Use:

SCANLAN® EASY-TAG™ is intended to be attached to sterilization trays, storage containers, instruments, devices or any items that need identification. SCANLAN® EASY-TAG™ is intended for single use only.

EASY-TAG™ Tracking System provides users with a means of identifying reusable surgical instruments' processing status or the contents of sterilization trays and allows placement of pre-printed labels or writing on both sides of the tag. The EASY-TAG™ Tracking System is intended to be used in steam, ethylene oxide (EO), or vaporized hydrogen peroxide (gas plasma) sterilization.

Instructions for Use:

Select the desired variety of EASY-TAG™. Apply desired writing, if applicable, to EASY-TAG™ prior to placement. Use elastomer loop to attach the tag to sterilization trays, storage containers, instruments, devices or any items that require identification. EASY-TAG™ may accompany its attached item through a single sterilization cycle.

Contraindications:

EASY-TAG™ is not intended to be reprocessed as it is a single use product.

EASY-TAG™ is not intended for use through a decontamination cycle.

EASY-TAG™ is not intended for any purpose other than identification and tracking.

EASY-TAG™ is not intended to become sterile.

Single-Use Risk of Reuse Statement:

Reprocessing or re-use of this SINGLE-USE DEVICE is prohibited. Any such reuse or reprocessing of this SINGLE-USE DEVICE shall void any warranty or liability by the manufacturer for injury or loss associated with inappropriate use. Reuse of single-use devices creates a potential risk of patient or user infections. Contamination of the device may lead to injury, illness, or death of the patient.

Requirements for Special Facilities, Training, or Particular Qualifications of the User and/or Other Persons:

Not Applicable

Methods for Safe Disposal:

The user is responsible to dispose of the device and packaging according to local and national requirements.

Incident Reporting Notice:

Any serious incident that has occurred in relation to the device should be reported to Scanlan International, Inc. and the competent authority of the Member State in which the user and/or patient is established.

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